CLEANING INSTRUCTION – Rescue Bags / Injured Protection Bags

Before cleaning the rescue bag all accessories (vacuum mattress, vacuum pump, blankets, etc.) have to be removed. These parts have to be cleaned according the cleaning instruction of the manufacturer of these parts.

In addition, the thermal insulation mats on the side and on the bottom (if available) must be removed before washing, for this purpose there are Velcro fasteners on the edges of the compartments.

Cleaning Instruction

- Hand-wash or hand-wash programme at max. 40°C (synthetic fibres, polyamide, polyester, polyacrylics).
- Clean blood or secretion spots with cold water as soon as possible.
- Air-dry, avoid direct sunlight.

- Do not chlorinate, do not apply any detergent containing chlorine.
- No dry-cleaning with hard brushes.
- No contact with disinfection baths or aggressive disinfects.
- Do not use metal tools to rubber spots of dirt.
- Do not use laundry driers or spin driers.
- No direct sunlight.
- Do not spin.
- Do not iron, as non-correctable changes are to be expected.

- For disinfection purposes, “ELTRA 40 Extra” from Ecolab has been tested and could be used.

Cleaning of the Velcro Fastener

- In addition to the above described washing procedure, dirt, forest remainders, etc. can and shall be removed from the Velcro fasteners by means of a soft brush (like hand-cleaning brush, etc.).

Dry all parts carefully after cleaning. No use of any laundry drier or no spinning!

Storage

- The product must be stored dry, protected from direct sunlight and at normal ambient temperature.
- The storage place for the rescue bag must be protected against and be free from any influence of all kind of chemicals which are able to harm textiles. On a heliport special attention shall be given to avoid contact with:
 - Fuels, lubricants, hydraulic oils and other technical liquids
 - Battery acids, gels
 - Disinfection materials of any kind
 - The fumes of these substances

- **If the rescue bag is wet, it must be carefully air-dried before storage. Never use a laundry dryer!**

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