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Operational and Maintenance Instructions

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Table of contents

The Individual Parts	2
Technical Data	3
Operational Applications of the UT 2000	3
Utilization as Back-Carrying Device	5/6
Utilization as Stretcher or Load Carrier	5/6
Utilization as Cot or Emergency Bed	7/8
Utilization as Sled in Snow	7/8
Parachute Jumps, Parachute Drops	9
Helicopter Rescue	10
Helicopter Rescue, Austrian AF	11
Helicopter rescue and Precipice Recovery	13
Checks for Operational Readiness & Condition	16
Maintenance and Cleaning	16
Regular Servicing, Storage	16
Uncoupling of two UT 2000	18
Accessories for UT 2000	19/20

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Back netting (11) storage
3 Helicopter lifting straps
2 pair load securing straps (13)

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Technical Data:**Utilization as Back-Carrying Device:**

without detachable frame	with detachable frame (standard)
Length: 940 mm	1 040 mm
Height: 120 mm	120 mm
Width: 440 mm	440 mm

Maximum permissible load: depends on the strength of the user.

Empty weight: 4,10 kg

Temperatura resistance -50°C to +100° C

Utilization as sled in snow/towing sled, load carrier, rescue unit, first-aid stretcher, emergency cot and helicopter rescue:

without / with detachable frame

Length: 1810 / 2000mm	Empty weight: 8,20 kg
Width: 440mm	Maximum permissible load:
Height: 120 / 220 mm	160 kg, 1 person

OPERATIONAL APPLICATIONS OF THE UT 2000

The UT 2000 can be employed for the following assignments either as a single device or when two devices are joined together:

- 1. BACK-CARRYING** device for transporting loads of all kinds: equipment, weapons, ammunition.
- 2. CARGO CARRIER** for loads up to 160 kg (2 UT 2000s coupled together).
- 3. STRETCHER** for transporting an injured person (2 UT 2000s coupled together).
- 4. SLED** in snow for the transportation of loads (1 or 2 UT 2000) or an injured person (2 UT 2000s coupled together).
- 5. PRECIPICE RECOVERY** (2 UT 2000s coupled together).
- 6. HELICOPTER SUSPENSION** for loads up to 160 kg, respectively for an injured person (2 UT 2000s coupled together).
- 7. COT, EMERGENCY BED** (2 UT 2000s coupled together).

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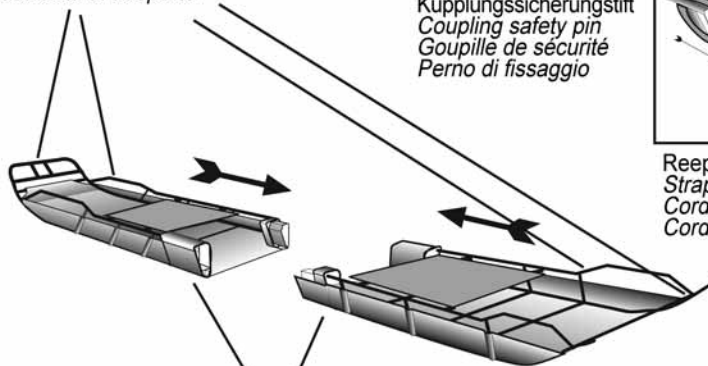
If using as a pullable sled, load carrier / first-aid stretcher, the two couplings must be secured via the securing pin or on the detachable tubes using a strap or rope.

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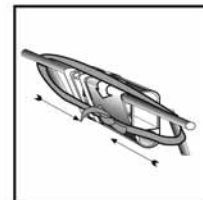
Utilization as Back-carrying**Device**

- * Insert the load frame (12) into a load frame positioner (3 choices) and secure.
- * Set the load on the frame and secure with load securing straps (13).
- * Release chest harness (6) and abdominal belt (8). Loosen centre of gravity straps (5) and height-adjustment straps (7).
- * Place UT 2000 on back-tighten abdominal belt (8), secure chest harness (6).
- * Adjust height-adjustment straps (7). Adjust centre of gravity straps (5) as necessary.

Utilization as Stretcher or Load Carrier (A)

- * Couple 2 UT 2000s together (9) (click into place-check by trying to pull it apart!)
- * **The two couplings must be secured via the securing pin or on the detachable tubes using a strap or rope.**
- * Secure the injured person or load into position with the hip straps (8) and the load securing straps.(13)
- * The Device can be carried by two or by four people.

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Utilization as Cot or Emergency Bed (B)

- * Insert detachable tubular frame (12) as foot rest.

Utilization as Sled in Snow (C)

- * Couple together 2 UT 2000s (9) (click into place - check by trying to pull apart!).
- * **The two couplings must be secured via the securing pin or on the detachable tubes using a strap or rope.**
- * Secure the injured person or load into position with the hip straps (8) and the load securing straps (13).
- * Secure the ski poles with rope or belts on the right side and the left side cross over in front of the sled for snow transportation.
- * **Hauling direction! Pay attention to the arrow on the sliding surface.**
- * For smaller loads 1 UT 2000 may be used.
- * Ackja attachment for UT 2000: see page 19/20 - Accessories.

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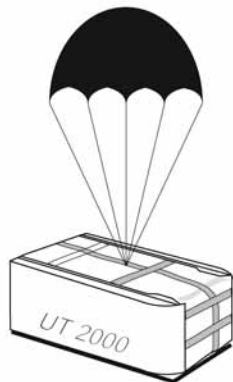
Cargo Parachute:

- * The UT 2000 is too light to be parachuted as a single load. This means that one or more UT 2000s are required to be attached to a load.
- * Each UT 2000 is secured to both sides of the load ready to be dropped, this being effected by means of the load securing straps, in which case the sliding surface must face outwards. The load frames must be dismantled and tightly lashed.

Caution! UT 2000 must be fastened as high as possible. The load must not be allowed to land on a UT 2000.

Welded seams according to ÖNORM M 7812, Grade 1/ Quality assurance according to AQAP 4 / 130, Tested for flight suitability by the Federal Office for Civil Aviation FS 3 / 88, issue 3. Alteration made in the Helicopter Suspension Harness approved by the Austro-Control Ltd. Zl. 8421-1/15-95. Flight Authorization of the German Military N 1670-13 of 15. 11. 91. Reg. Nr.6545-2005-01-Issue 1

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Helicopter rescue and Precipice recovery

- * Couple two UT 2000s together in such a way that both pairs of couplings fit into one another easily and that the sliding surfaces overlap. Push both UT 2000 together until the spring units on the couplings can be heard clicking into place; check each coupling by pulling apart and insert the securing pin.
- * Remove the suspension straps black - at the head and feet - and suspension strap yellow - at central body level from each of the two back netting pockets. (page 2, pos.11)
- * Check all suspension straps for damage, e.g. cuts, broken threads, corrosive substances, acid, fuel, traces of burrs or deformation. Replace any straps that are found to be defective.

Suspension straps: Utilization as helicopter outer straps 15 years from the month of manufacture (each suspension strap is fitted with a label indicating the date of manufacture).

A utilisation duration of 10 years is recommended for textile straps.

- * Attach respectively a black suspension strap (S) to the lower tube of the tubular frame by threading one end of the suspension strap through the twisted loop; the strap of the head side should however be on the outer side of the carrying handles.
- * Pass one yellow suspension strap along the couplings. The ends of the straps must be passed round the first cross strut and the frame.

Information German Army:

Attach a yellow suspension strap (Y) to the lower tube of the tubular frame on the left and right side of the coupling (see picture on Page 10 Fig. A) by threading one end of the strap through the twisted loop.

- * For combining and hooking in the suspension straps, use must be made of mountaineer's safety hooks (Karabiner) with a minimum breaking load of 2200 daN (e.g. HMS spring hooks). (Karabiner)

Following suspension methods are possible for helicopter winch rescue:

Both black loops and both yellow loops of the suspension straps are brought together on one side and hooked into a screw hook. Both screw hooks are brought together in the supporting plate, steel ring or steel spring hook and placed in the winch hook.

The way that this is inserted in the winch load hooks on the helicopter depends on the safety and aeronautical regulations governing the type of helicopter in use.

The flight rescuer hooks himself into the central load hook (UT at hip height).

- * After the mission has undergone completion, return the suspension straps to the back netting pockets.
- * Maximum speed as helicopter-suspended load with loaded rescue device 80 km/h; only hovering is possible in the no-load state, without the security of a flight assistant.

- * **Caution!** Injured persons or transported loads are secured by means of hip straps (page 2, pos. 8) and load securing straps (page 2, pos.13). When transporting persons, the arms must be fixed in place!
- * **Caution!** The safety regulations governing helicopter transport must be strictly adhered to (see Rotocraft Flight Manual Supplement packed inside the back netting pockets of each UT 2000.)
- * Raising / lowering an injured person with the winch must be carried out with an antirotation cable with predetermined breaking load of 50 kp (rated breaking cord).

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Checks for Operational Readiness and Condition

The operational readiness check of the UT 2000 includes a visual examination for damage and a control to ensure that the device is complete and intact.

Parts to be checked:

- * Device tubular frame and detachable frames for cracks, breaks and corrosion.
- * Insertion possibilities of the detachable frame. (12)
- * Coupling function. (9)
- * Sliding surface (1) and sliding surface joints (1) for distortion, cracks or breaks.
- * Correct seating of all rivet joints.
- * Date of expiration of the suspension straps.
- * All strap edges for cut, melted and worn spots.
- * Seams of the suspension straps for intactness.
- * Straps for spots which show damage of the fabric or other damages (corrosive materials, acid, adhesives, fuels, traces of burls or compressed parts).
- * Completeness of the strap material.
- * Legibility of the operating instructions plate. (2)
- * Recognizability of the directional arrows. (9)

Regular Servicing

- * Maintenance and Cleaning after each use.
- * Check for operational readiness and condition before each use as well as at annual intervals.

Maintenance and Cleaning.

A damp device must be dried and well aired. Soiled straps should be brushed clean with clear water or a mild detergent. Do not use any other chemical cleaners. If the device has been in contact with salt water it should be cleaned immediately with clear fresh water.

Attention: The device should not be allowed to dry in direct sunlight or close to a direct heat source.

Corrosion traces on the rivet joints shall be removed with a polishing linen cloth and then lubricated with weapon oil.

Storage : UT 2000s are to be stored in dry rooms, away from direct sunlight and protected from the weather, so that the strap material is kept ready for use at all times.

Attention: The UT 2000 may only be stored in a dry condition!

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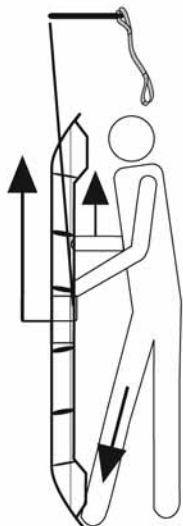
Uncoupling of 2 UT 2000

Coupling with built-in quick-opening system.

If using a coupling safety pin, this pin must be released.

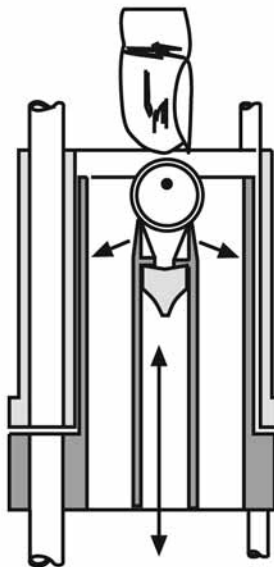
Hold the lower UT 2000 with one foot, pull the upper UT 2000 slightly upward with one hand. Press with the other hand the quick opening device built-in in the coupling and pull the device upward. (See figures).

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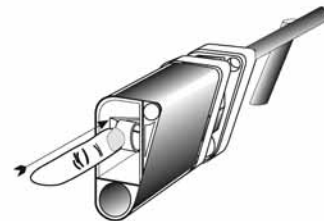
Extension handle for wheel and ackja-attachment.

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Ackja attachment alpin / one-man system

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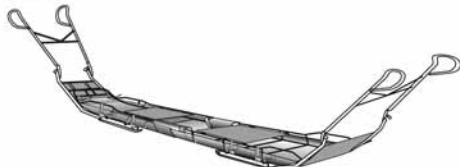
Ackja attachment alpin / two-man system

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
Wheel attachment alpin / one-wheel system.

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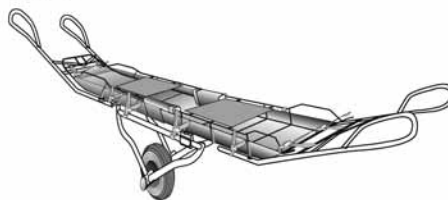Radzubehör Alpin / Zweiradsystem.

Wheel attachment alpin / two-wheel system.

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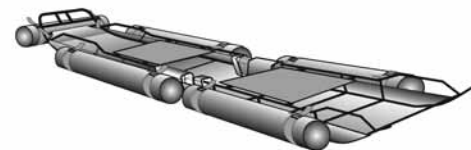
Floating collar attachment for water and canyon rescue.

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Helicopter suspension straps with variable attachment possibilities.

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Left and right lateral protection frames for protecting the uninjured person and widening the lying surface.

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Large frame with transparent protection shield with handle on the UT.

Also adapted for carrying heavy loads.

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Antirotation system for raising and lowering an injured person with winch.

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The adjusting belt for the helicopter straps is used to balance the lying position of the injured person and to secure in the helicopter.

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Steel skids with adjustable handles and adaptable wheel system on the UT-. Utilisation as pullable sled for summer and winter use.

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Rotorcraft Flight Manual Supplement for Personal Rescue Litter UT-2000

This Supplement shall be attached to the pertinent Rotorcraft Flight Manual when personal rescue is intended with this equipment.

**Muster
Bestellnummer
42030030**

Statement-Aeronautical Compatibility
WTD 61 – type testing of aviation equipment for the German Army
Reg.Nr.: 6545-2005-01 of January 14, 2005
Issue: 1

1. General

The personal rescue litter UT-2000-Dual consists of two parts of the UT-2000 which are connected together with quick locking devices as described in the appropriate equipment manual. The litter is suspended to the helicopter by its belts, a swivel and a certificated recovery tow to the cargo hook or direct on the hoist. The transport must be assisted by an external crew member, also hooked to the helicopter.

2. Limitations

Max. speed with empty litter: hovering only

Max. speed with occupied litter: 80 km/h

Max. weight of payload: 160 kg

The electric latch disconnect and the cable cutter must be inactive

3. Normal procedure

The patient will be carried in the litter and the transport must be assisted by a crew member.

A full description and maintenance instructions are given in the pertinent equipment manual.

4. Emergency

In case of an emergency the empty litter can be removed from the helicopter by disconnecting with the manual release.

In case of an emergency with an occupied litter the litter and the external attached crew member should be dropped from the helicopter to ground with greatest possible care by manually releasing the connection.

Kohlbrat & Bunz GmbH
A - 5550 Radstadt, Austria, Europe
phone +43 - (0) 6452 - 7193 0, fax +43 - (0) 6452 - 7193 51
E-.mail: office@kohlbrat-bunz.com
www.kohlbrat-bunz.com

Internationale Patentanmeldungen
International Patents Registered
Demandes internationales de brevet
Certificati di brevetto internazionali

Technische Änderungen vorbehalten.
Subject to technical changes.
Sous réserve de modifications techniques.
Modifiche tecniche attuabili senza preavviso.

Entwicklungsstand 2007- Development state 2007- Niveau de développement 2007- Innovazione aggiornamenti 2007. Technische Änderungen vorbehalten. Subject to technical changes. Sous réserve de modifications techniques. Modifiche tecniche attuabili senza preavviso.